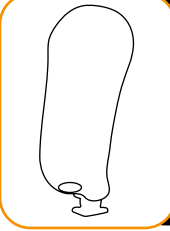
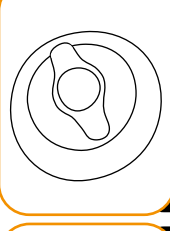
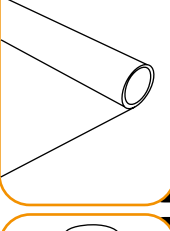
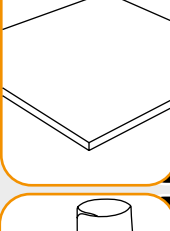
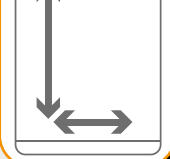
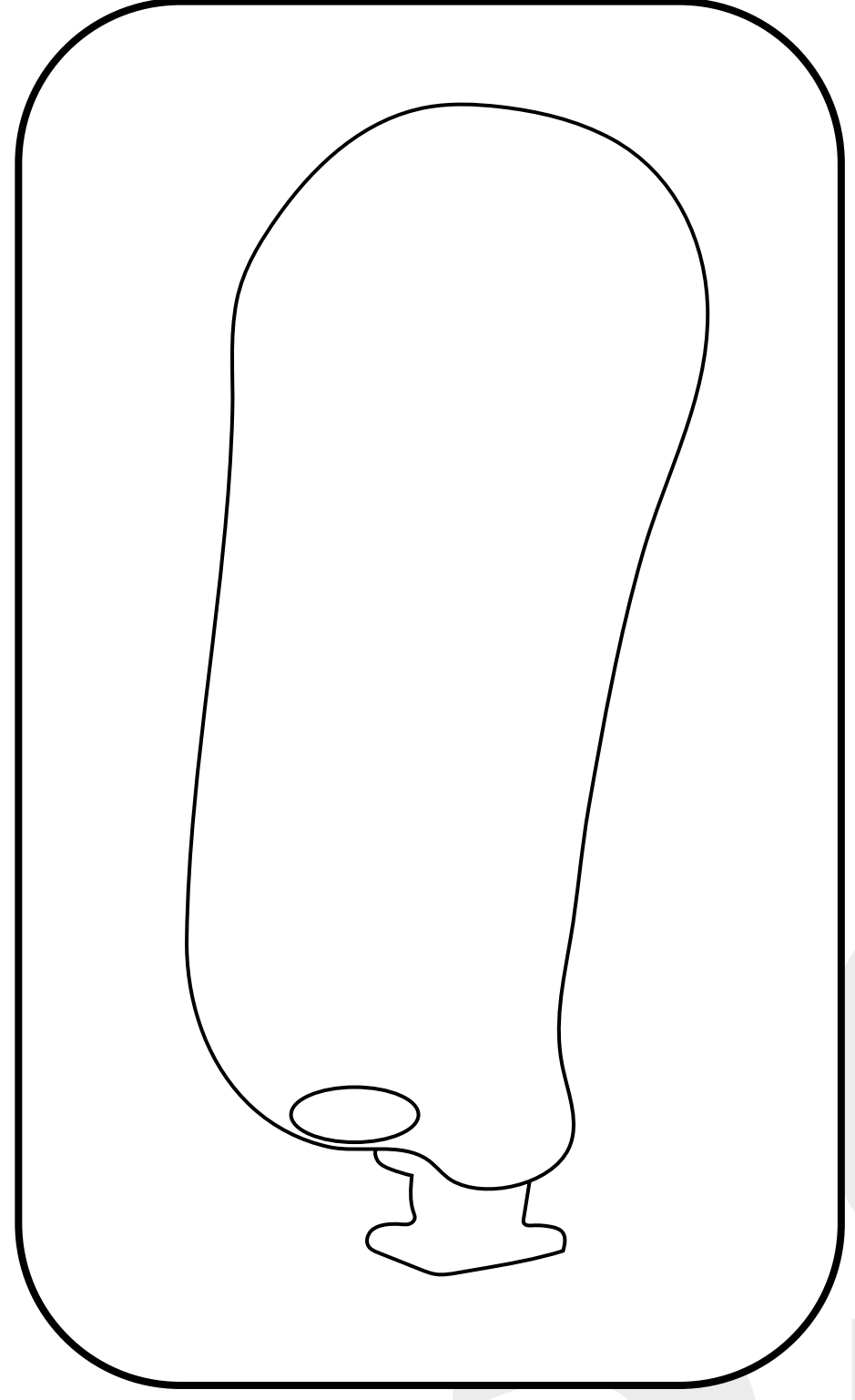
<p>Subischial MWK Schaft Subischial MWK socket</p>	<p>2 7</p>	 <p>Milwaukee TF Schaft</p>
<p>Prothesentechnik Prosthetic Components</p>	<p>8 25</p>	 <p>Prothesentechnik</p>
<p>Prepregtechnik PrePreg Technology</p>	<p>26 31</p>	 <p>Prepregtechnik</p>
<p>Silikontechnik Silicone Technology</p>	<p>32 37</p>	 <p>Silikontechnik</p>
<p>Verbrauchsmaterialien Materials</p>	<p>38 55</p>	 <p>Verbrauchsmaterialien</p>
<p>Servicefertigung Service Fabrication</p>	<p>56 95</p>	 <p>Servicefertigung</p>
<p>Bestell-/ Maßformulare Ordering-/ Dimension Forms</p>	<p>96 136</p>	 <p>Bestell-/Maßformulare</p>

Subischial MWK Schaft
Subischial MWK Socket

2
7

Milwaukee TF Schaft

Prothesentechnik

Prepregtechnik

Silikontechnik

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Servicefertigung

Bestell-/Maßformulare

Milwaukee TF Schaft

Prothesentechnik

Prepregtechnik

Silikontechnik

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Servicefertigung

Bestell-/Maßformulare

250

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MILWAUKEE TF SCHAFT®

Hintergrund

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Background

Scientifically developed sub-ischial socket system, which has established and proven itself on the German market since 2015. It is listed as a reimbursable aid by both private and public health insurance companies and the Federal Guild Association and is billable by prosthetics providers upon certification.

Certification and quality assurance

Fitting a MWK socket® is only possible for certified prosthetics provider businesses in conjunction with a specifically trained/certified practitioner. Our advanced training system is characterized by its exacting standards and is the basis for the sustained high quality of MWK fittings. To further ensure product quality, each MWK fitting undergoes a quality check.

Reimbursement of costs by health insurance companies requires

- :: The Serial number for the respective socket system
- :: MWK certification of the prosthetist/business
- :: certification of the quality inspection.

For reasons of quality assurance and intellectual property rights no non-certified businesses may offer the MWK system.

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Each Milwaukee TF Socket® receives a certificate and seal with a unique serial number. The certificate serves as proof for insurers/payers and indicates original quality of care.

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Indication

Indicated for trans-femoral amputees of activity levels 2 through 4, with muscle status of 3 through 5 (after Smith) and a residual length from short to long. Soft tissue coverage is irrelevant. May replace any prior prosthesis socket design; quadrilateral, ischial containment, or anatomical / MAS socket shape.

Benefits

- :: Clear trend of mobility increase after new fittings
- :: No bony contact areas
- :: Socket system is based on a pure muscle guidance
- :: Improved load bearing in the stance phase
- :: Low trim lines
- :: Optimized hip loading through physiologic transmission
- :: Lightweight yet highly stable
- :: Easy to clean and maintain for the patient
- :: Cosmetically inconspicuous
- :: Full hip range of motion

Variants / please note

- :: Custom validation by Guenther Bionics® for each patient fitting
- :: Only available for certified customers or through fittings in direct collaboration with Guenther Bionics®
- :: Optional: non-stick coating

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Certification guideline MWK® System

The Milwaukee TF socket® has matured in recent years thanks to intensive research in the field of transfemoral amputation and the parallel ongoing technical development in prosthetic technology and has now grown into a well-established socket concept. In order to ensure that the specialists in our industry are provided with the right training framework when adopting the MWK socket in their practice, a complex certification system is essential. The quality of a prosthesis always begins with the individual socket, which is the biggest and most difficult task in prosthetic fitting. A practitioner has to face this challenge again and again and that is only possible with a constant exchange of knowledge between experts.

The MWK certification system has a multi-layered structure to give you the best basis for future success with the new socket concept. The first step is to attend a basic seminar. There, the basic theory is conveyed and trial fittings are carried out, analyzed, and optimized with your own patient(s). In the further course, two additional fittings supervised by Guenther Bionics® instructors must be carried out in the so-called Cert phase after which the MWK Training Certificate is issued. In order to continuously ensure and optimize the quality of care, at least one pro seminar is offered annually.

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:: 3 day seminar under the direction of Dipl. Ing Michael Günther

:: Contents:

Biomechanics and function principles of the sub-ischial socket

Plaster casting and modifications

Check socket production and fitting to the patient

:: Participation with 2 practitioners/technicians + 1 own patient or test patient

3. After the basic seminar:

:: Support by Guenther Bionics® for subsequent MWK fittings

(Free of charge)

:: Certification as a qualified provider

Certification MWK socket®

:: Gain practical experience

:: Use Guenther Bionics support (free)

:: Carry out and successfully complete 2 accompanying fittings independently under QM with Guenther Bionics

After successful completion, receive a certificate as MWK

socket® - certified business

Pro seminar MWK socket®

seminar content:

:: Exchange with other certified practitioners/businesses

:: Presentation of current and planned scientific studies

:: Access to the network of research and development of the participating universities and Guenther Bionics®

:: Participation in this network

:: Maintaining certification

:: Option to obtain MWK-Socket Center of Excellence status with the applicable privileges

Milwaukee TF Schaft

Prothesentechnik

Prepregtechnik

Silikontechnik

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Servicefertigung

Bestell-/Maßformulare

Milwaukee TF Schaft

Prothesentechnik

Prepregtechnik

Silikontechnik

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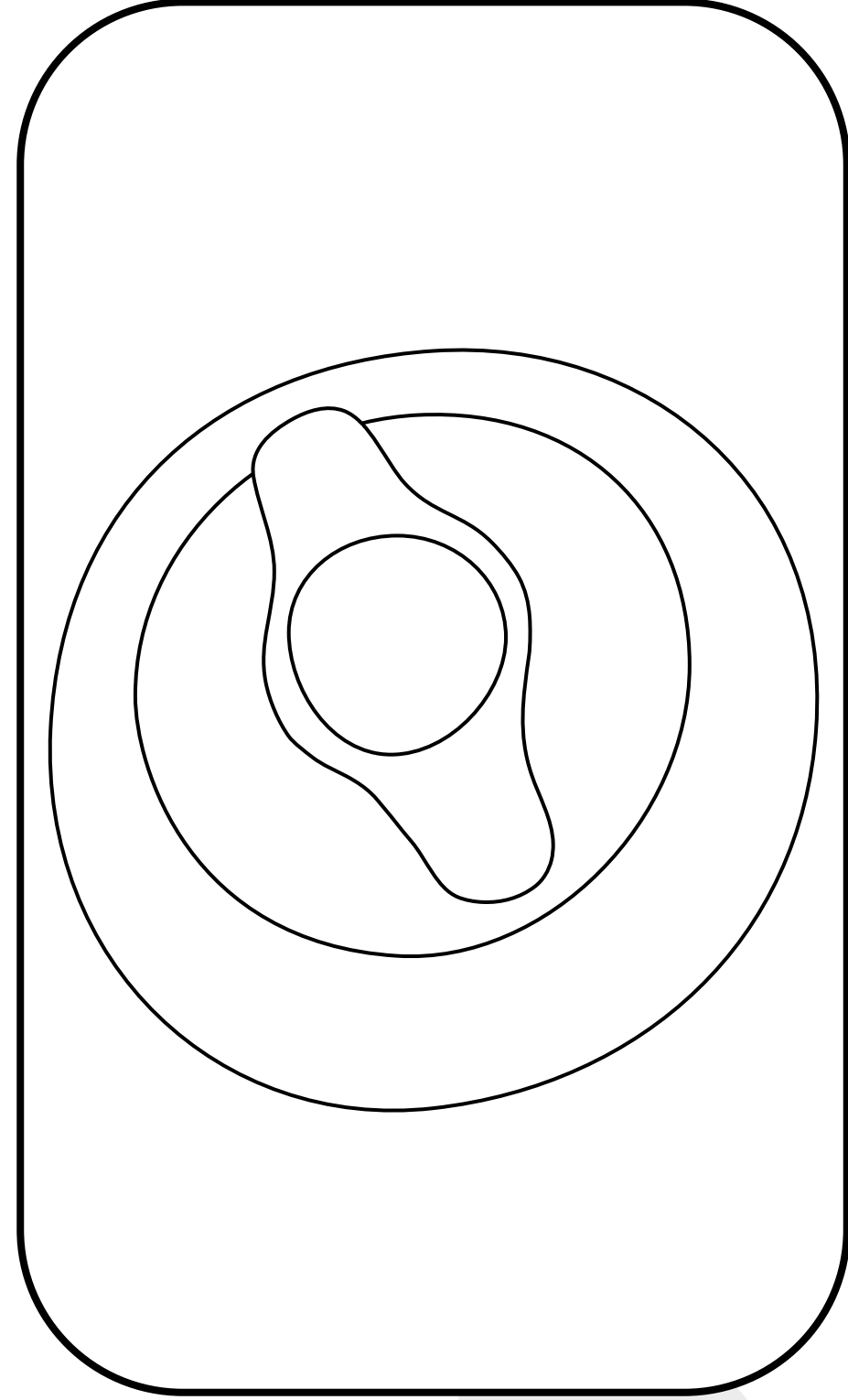
Servicefertigung

Bestell-/Maßformulare

Prothesentechnik Prosthetic Components

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Flex Series

The Flex Series lower leg covers feature a waterproof, one-piece, lightweight and flexible design. The cover is particularly easy to remove from the joint, so it is not an obstacle for charging or cleaning. Full materials are used for the cover, resulting in good shock padding. In addition, the knee joint is lightly padded ventrally by the cover in the kneed position.

The Flex series includes three different versions of the cover.

The **FlexProtector -Standard-** (4K03=1) serves as protection and can be ordered for any standard knee joint.

The **FlexCover -Individual-** (4K03=2) is made to measure and extends to just before the foot cosmetic. The length of the cover is optimally adapted to the needs of the fitting.

The **FlexCover -Premium-** (4K03=3) can be ordered on request and is manufactured according to the patient's wishes after scanning. The cover extends to the foot cosmetic and can be adapted to any individual shape of foot cosmetic.

Advantages

- :: Easy and fast order processing
- :: Waterproof
- :: Flexible and very light material
- :: 100% reproducibility, therefore a new production due to wear is associated with very low costs

Design variants

- :: Individual coloring possible
- :: All commercially available knee joints can be covered by the Flex series
- :: Individual shaping possible on request

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Trans-femoral prosthesis socket in SilPreg® technology

This system is used in trans-femoral prosthetics. It is characterized by a very low weight and excellent skin compatibility of the inner socket, made possible by the use of modern materials. The design is tailored to the customer and can be individualized for each patient. The inner socket material is medical grade silicone. Silicone is permanently and inherently stable and can be easily and conveniently cleaned.

Benefits

- :: Improved hygienic properties and good disinfectability
- :: Very well suited for patients with skin allergies and sensitive skin
- :: Effective relief of pressure from bony prominences or exostoses
- :: Silicone inner socket maintains its shape and is not susceptible to developing odors
- :: Very good adhesion properties

Suitable for K levels 2-4 for everyday prostheses, sports prostheses and bath prostheses.

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Trans-femoral prosthesis socket in SilPreg® technology

This system is used in trans-femoral prosthetics. It is characterized by a very low weight and excellent skin compatibility of the inner socket, made possible by the use of modern materials. The design is tailored to the customer and can be individualized for each patient. The inner socket material is medical grade silicone. Silicone is permanently and inherently stable and can be easily and conveniently cleaned. During manufacturing, reinforcements, gel padding, and zones of different Shore hardness can be implemented. Likewise, a variety of color and latching options are available.

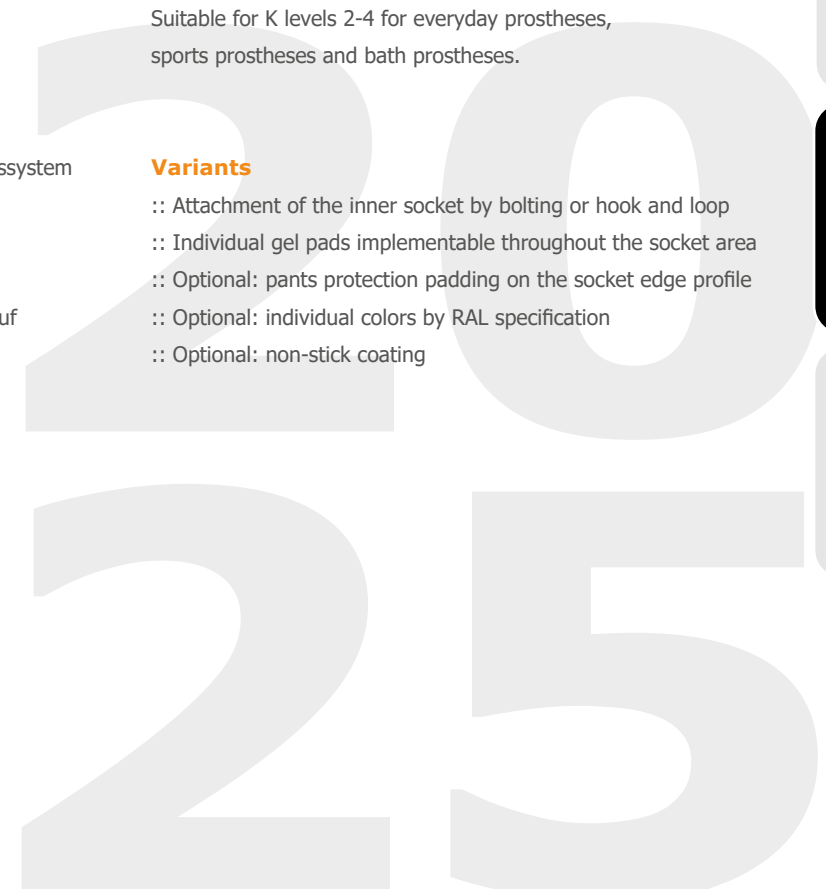
Benefits

- :: Very good fitting option for residuals with complicated scar conditions
- :: Improved hygienic properties and good disinfectability
- :: Individual bedding options for pressure-sensitive regions of the residual limb
- :: Very well suited for patients with skin allergies and sensitive skin
- :: Effective relief of pressure from bony prominences or exostoses
- :: Silicone inner socket maintains its shape and is not susceptible to developing odors
- :: Very good adhesion properties

Suitable for K levels 2-4 for everyday prostheses, sports prostheses and bath prostheses.

Variants

- :: Attachment of the inner socket by bolting or hook and loop
- :: Individual gel pads implementable throughout the socket area
- :: Optional: pants protection padding on the socket edge profile
- :: Optional: individual colors by RAL specification
- :: Optional: non-stick coating

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Swim prosthesis socket TF Flexible in SilPreg® technology

This system is used in sport and swim prostheses. It is characterized by its water resistance, easy cleaning and excellent skin-friendliness, made possible by the use of modern materials.

The design is tailored to the customer and is individualized for each case. The material used is medical grade silicone, which can be simply and easily cleaned and retains its shape. In the manufacturing process, reinforcements, gel padding and zones of different Shore hardness can be implemented. Likewise, a variety of color options is available.

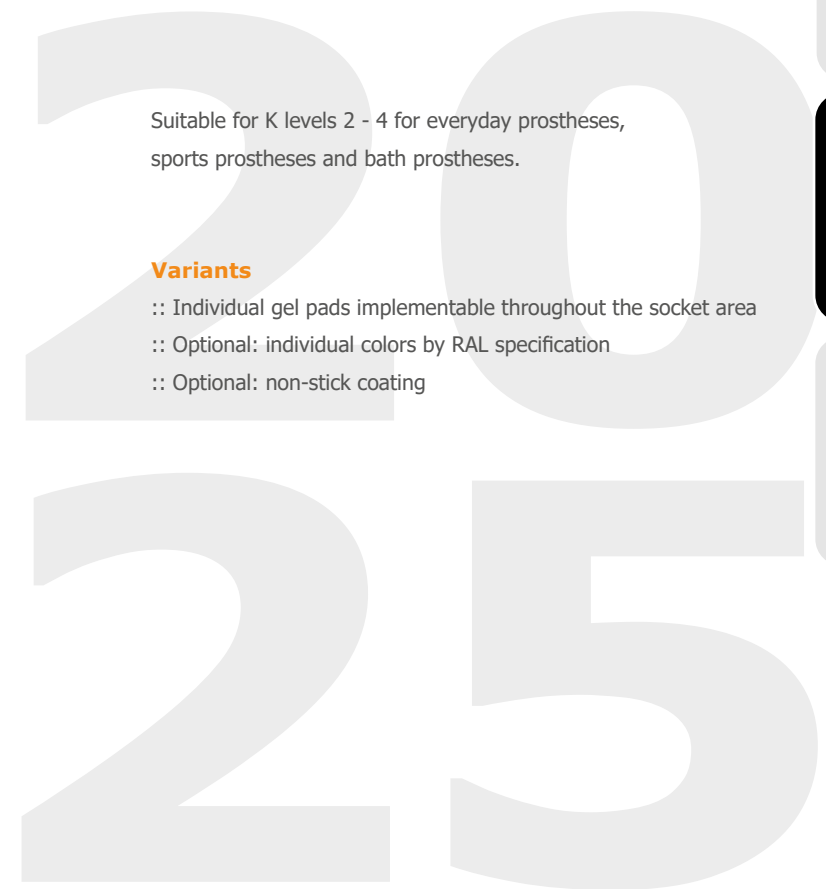
Benefits

- :: Waterproof through a complete sheath of silicone
- :: Very easy to clean
- :: Individual bedding options for pressure-sensitive limb regions
- :: Very good option for complicated scars
- :: Improved hygienic properties and good disinfectability
- :: Very well suited for skin allergies and sensitive skin
- :: Effective relief of bony prominences or exostoses
- :: Silicone socket remains permanently dimensionally stable and is not susceptible to odors
- :: Very good adhesion properties

Suitable for K levels 2 - 4 for everyday prostheses, sports prostheses and bath prostheses.

Variants

- :: Individual gel pads implementable throughout the socket area
- :: Optional: individual colors by RAL specification
- :: Optional: non-stick coating

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Milwaukee TF socket®

Trans-femoral socket fittings commonly are based mainly on three different techniques / shapes. These are the most well-known form of the quadrilateral socket, along with the more recent ischial containment socket, and the current state-of-the art MAS or anatomical socket. All three socket systems utilize bony support structures as well as partially bony locks to transfer occurring forces (body weight) as painlessly as possible from the amputation residual into the prosthesis. This inevitably leads to improper loading of the spine and the pelvis and results in skin irritation in the area of bony prominences.

Intensive research in the field of trans-femoral prosthetics and the parallel technical development in orthopedic technology has helped mature the MWK Socket® into a socket concept that eliminates the necessity of bony support areas and thus enables optimized physiological load to promote patient activity and to reduce comorbidities due to incorrect posture. Unpleasant skin irritations on bony contact surfaces are avoided, which allows the fitting even of patients with load-sensitive bony structures.

According to preliminary study results, the system greatly improves social integration and participation as users state that they can cover longer walking distances and their average walking speed increased significantly.

What are the design features of the MWK Socket®?

The new shaft system is approved **exclusively** in the format of the special definitive socket 1S011=1 / 1S011=2 / 1S011=3, which thanks to the combination of silicone gel padding, matrix and socket shape ensures the desired improvements. Due to the muscular containment the load on the os ischium is greatly reduced or entirely eliminated, which in turn relieves the sacrum / lumbar spine. Hip range of motion is nearly unrestricted due to the low socket brim cut lines. Thus sufficient space is provided to not tense up the muscles and ligaments of the affected side. In addition, the system uses a special inner socket from medical grade silicone, the adhesion properties of which are far superior to conventional materials. The material can be autoclaved/disinfected, is dimensionally stable and durable, and its material properties are very similar to the skin's. In addition, a separate gel padding and matrix assembly is included in the muscle tendon area, whereby the pure muscle guidance is made possible.

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Knee disarticulation socket in SilPreg® technology

This system is used in knee disarticulation and long trans-femoral prosthetics. It is characterized by a very low weight and excellent skin compatibility of the inner socket, made possible by the use of modern materials. The design is tailored to the customer and can be individualized for each patient. The inner socket material is medical grade silicone. Silicone is permanently and inherently stable and can be easily and conveniently cleaned. During manufacturing, reinforcements, gel padding, and zones of different Shore hardness can be implemented. Likewise, a variety of color and latching options is available.

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Hip prosthesis socket in SilPreg® technology

System for use in hip disarticulation prosthetics. The advantages of modern materials combine the excellent rigidity of the carbon hip basket with the high comfort and ease of use of the flexible silicone inner socket. Different designs are possible, to assure a solution tailored to your and individual patient's needs. The medical grade silicone can be cleaned easily and comfortably, and it permanently retains its dimensional stability and elasticity.

Typical customizations include selective structural reinforcements, embedded gel cushions, material hardness gradations, color options, and adapter options.

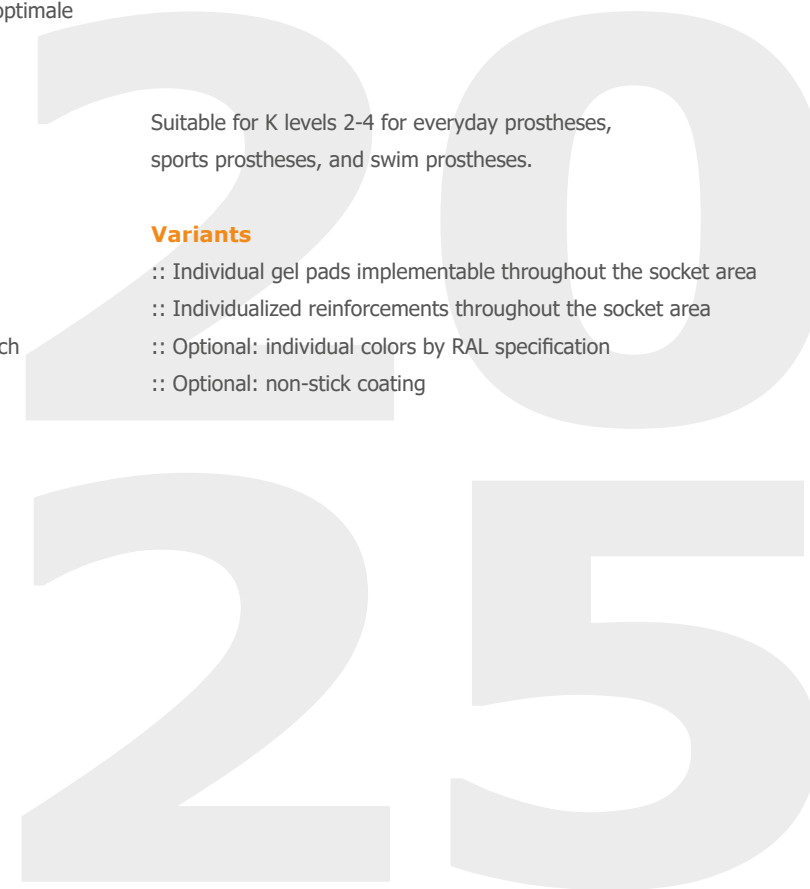
Benefits

- :: Very good fitting opportunities for hip dislocation and ultra-short trans-femoral amputations
- :: Excellent hygienic properties and ease of use
- :: Comprehensive options for embedding of pressure-sensitive regions and bony structures
- :: Very good material compatibility even with allergies and sensitive or irritated skin
- :: Stable pelvis containment and prosthetic suspension, while relieving pressure on sensitive bony structures
- :: Hygienic and mechanical properties of the silicone inner socket for sustainable fitting quality
- :: Very good adhesion of the material for optimum prosthetic suspension and control

Suitable for K levels 2-4 for everyday prostheses, sports prostheses, and swim prostheses.

Variants

- :: Individual gel pads implementable throughout the socket area
- :: Individualized reinforcements throughout the socket area
- :: Optional: individual colors by RAL specification
- :: Optional: non-stick coating

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Forearm prosthesis socket in SilPreg® technology

Like all of our prepreg / silicone products, this system provides a low overall weight combined with high skin-friendliness of the inner socket. For the application in trans-ulnar prosthetics, this allows optimum strength with a minimalist design. Generous openings allow elbow mobility, unimpeded by the carbon outer socket, as well as efficient and user-friendly integration of myoelectric components. The dimensional stability and resistance to contamination of the medical grade silicone material promise first-class comfort and excellent functionality.

Shore hardness, padding, reinforcements, colors, and adapter options are tailored to your needs and the needs of your patients.

Benefits

- :: Very good fitting options even for unusual shaped, or scarred residual limbs
- :: Rigidity of the outer socket in small steps selectable
- :: Perfect and permanent contact between the inner and outer socket
- :: Hygienic properties and disinfection friendliness of the silicone inner socket allows use for patients with skin allergies and sensitive skin
- :: Good adhesion properties even with cutouts for Myo-electrodes
- :: Easy modular adaptation by integrated terminal adapter

Suitable for K levels 2-4 for everyday prostheses, sports prostheses and swim prostheses.

Variants

- :: Individual gel pads implementable throughout the socket area
- :: Individualized reinforcements throughout the socket area
- :: Optional: individual colors by RAL specification
- :: Attachment of inner socket by bolting or hook and loop system
- :: Optional: non-stick coating

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Upper arm prosthesis socket in SilPreg® technology

Our trans-humeral socket prosthetic system is based on the same advanced materials prepreg carbon HTV and silicone that have been adapted for the demands of this application. Lightweight and hypoallergenic, the primary characteristics of this technology will lead to greater comfort, better prosthesis control, and pleasing visual appeal. The higher utility of the prosthesis is maintained over time by the wear-resistant and hygienically disinfectable medical silicone of the inner socket. The illustrated model is an example of the many available options that we have developed individually to meet your needs for your patients.

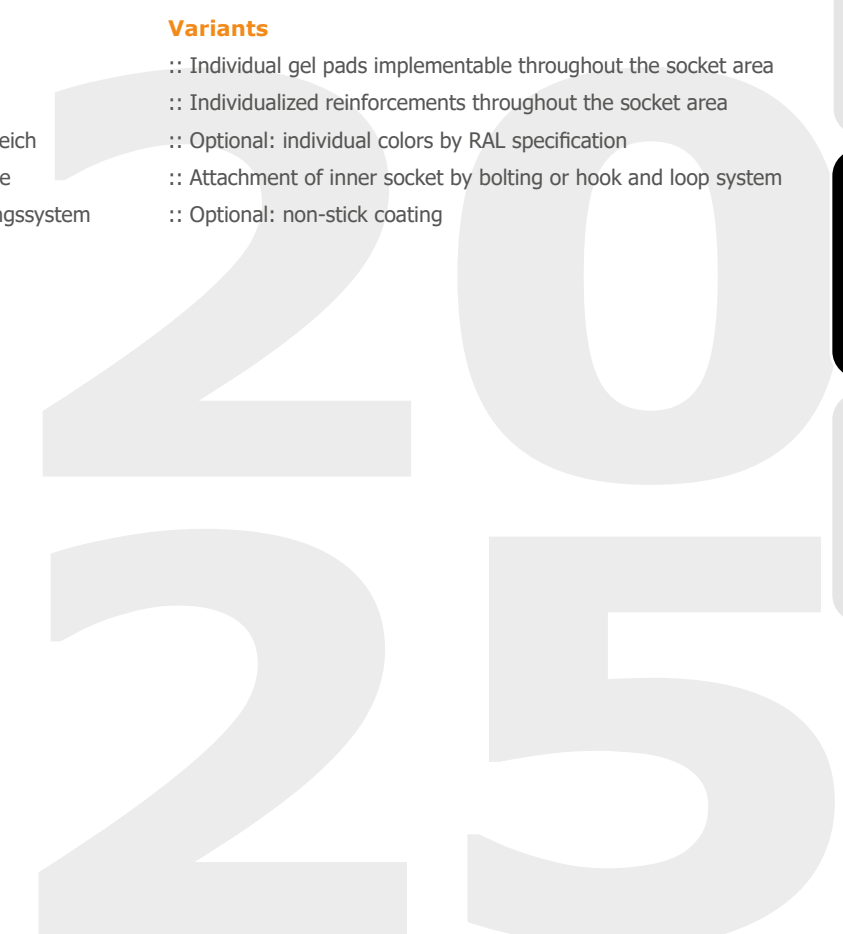
Benefits

- :: Very good fitting options regardless of residual limb dimensions and -shape
- :: Excellent hygienic properties
- :: Material compatibility also with skin allergies and sensitive / irritated skin
- :: Wear resistance, shape stability, robustness against stains, dirt and odors
- :: Perfect prosthesis suspension and guidance thanks to very good adhesion of the silicone material

Suitable for K levels 2-4 for everyday prostheses, sports prostheses and swim prostheses.

Variants

- :: Individual gel pads implementable throughout the socket area
- :: Individualized reinforcements throughout the socket area
- :: Optional: individual colors by RAL specification
- :: Attachment of inner socket by bolting or hook and loop system
- :: Optional: non-stick coating

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Shoulder prosthesis socket in SilPreg® technology

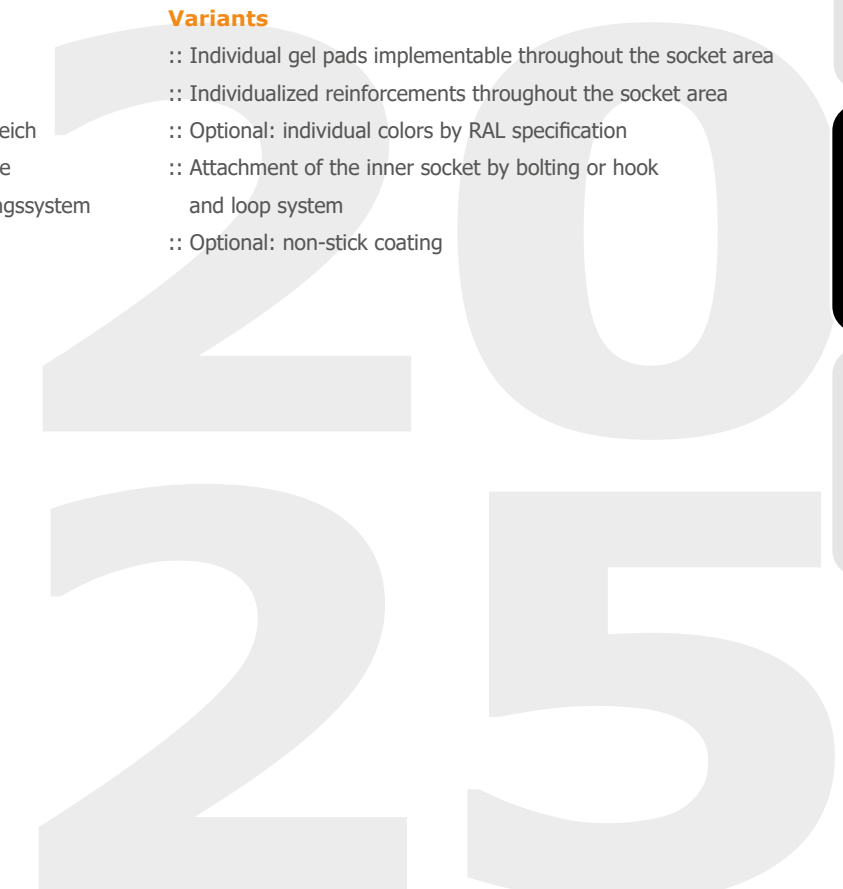
The shoulder prosthesis socket is another application that benefits greatly from the now available SilPreg® technology. The specific requirements of shoulder disarticulation prosthetics are met by the excellent material strength of the outer socket, and the high adhesion and skin-friendliness of the inner socket. Myoelectric prosthesis control electrodes may be incorporated during fabrication, as most any of your client wishes can be considered as well. Reinforcements, gel cushion, zones of different hardness, colors, connection options and the like are available in a wide selection.

Benefits

- :: Fitting of shoulder disarticulations, ultrashort trans-humeral residuals and arm defect formations
- :: Very good adhesion properties require minimal contact area
- :: Efficient design prevents movement restrictions and excessive perspiration
- :: Unproblematic hygienic properties, the inner socket can be cleaned and disinfected separately
- :: Good skin tolerability even with unusual / badly healed scars
- :: No material aging, silicone properties of stability and elasticity remain permanently

Variants

- :: Individual gel pads implementable throughout the socket area
- :: Individualized reinforcements throughout the socket area
- :: Optional: individual colors by RAL specification
- :: Attachment of the inner socket by bolting or hook and loop system
- :: Optional: non-stick coating

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Thumb abduction orthosis from silicon

This orthosis is primarily used in the stabilization of the thumb and the thumb base joint to improve the grip and hold function.

The design is tailored to the customer's needs and is individual for each patient. The material used is medical grade silicone, which can be cleaned easily and comfortably and remains very stable in shape.

During fabrication, volar reinforcements, different Shore hardness, colors, and hook-and-loop closures are implementable.

Indications

- :: Spastic paralysis (ICP)
- :: Rheumatoid arthritis
- :: Cosmetically inconspicuous prescription for low degree Thumb-in-palm deformity

Variants

- :: Hook-and-Loop closures with or without pulley
- :: Optional: Transition function zones by textile coating
- :: Optional: non-stick coating

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Forearm orthosis from silicone

This orthosis stabilizes the wrist and the basal / carpo-metacarpal joint. Edge design and joint position can be customized for each patient according to the customer's specifications. The medical silicone material can be cleaned easily and effectively, and keeps its shape and elasticity.

Volar reinforcements made of silicon or carbon prepreg, zones of different hardness, and hook and loop closures can be implemented in fabrication. Individual colors are also possible.

Indications

- :: Spastic paralysis (ICP-supplies)
- :: Flaccid paralysis
- :: Rheumatoid arthritis
- :: Cosmetically inconspicuous prescription for low degree Thumb-in-palm deformity

Variants

- :: Hook-and-Loop closures with or without pulley
- :: Optional: Transition function zones by textile coating
- :: Optional: non-stick coating
- :: Optional: Gel pad

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Foot orthosis

This orthosis is designed primarily for use in the stabilization and compression of the entire foot and for the application of muscle stimulation therapy. Flexible manufacturing processes allow for the consideration of individual circumstances and customer requirements in fabrication. The used material, medical grade silicone, is durable and retains its shape and elasticity. Material thickness, strength, and color can be modified individually during manufacturing. Likewise, silicone or prepreg reinforcements, zones of different hardness, and closures may be incorporated.

Indications

- :: Muscle stimulation
- :: Slight ankle instability
- :: Burns / scars compression

Variants

- :: Hook-and-Loop closures with or without pulley
- :: Optional: Transition function zones by textile coating
- :: Optional: non-stick coating

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ICP sandal in SilPreg® technology

This orthosis for active patients is mainly used for stabilization and compression of the entire foot and for muscle stimulation. The plantar area is adjusted to enable use without footwear.

Details of the design are freely modifiable and can be tailored individually to each patient. During the production silicone or carbon reinforcements are implemented. Shore hardness and color of the material is freely selectable, as is size, number and arrangement of the hook and loop fasteners.

Indications

- :: Muscle Stimulation (ICP- treatment)
- :: Flaccid paralysis / foot dorsiflexion
- :: Injuries of the ankle joint capsule
- :: Ligament injury or instability
- :: Ankle instability

Variants

- :: Hook-and-Loop closures with or without pulley
- :: Optional: Transition function zones by textile coating
- :: Optional: non-stick coating

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Silicone partial foot prosthesis in SilPreg® technology

This system is used in the prosthetic care after forefoot amputation. It is characterized by an excellent skin compatibility and high flexibility, made possible by the use of modern materials.

The design is tailored to the customer and can be individualized for each patient case. The material used is easily cleaned, shape retaining medical grade silicone. In manufacturing, reinforcements, gel padding and zones of different Shore hardness can be implemented. As always, a variety of color variations is possible to create a detailed replication of the forefoot.

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SilAir Partial foot prosthesis in SilPreg® technology

This system is indicated in the prosthetic care after forefoot amputation. It is characterized by a **novel compressibility**, excellent skin friendliness and high flexibility, made possible by the use of modern materials.

The design is matched to the customer's request and is individually customized for each patient case. The used material is medical silicone. It can be cleaned easily and conveniently and it stays dimensionally stable. Reinforcements, gel pads and zones of different shore hardness are implementable during manufacturing. Likewise, a multitude of color variations can be realized in order to create a detailed replica of the forefoot.

Benefits

- :: **Compressible forefoot**
- :: **Lighter than conventional silicone forefoot prostheses**
- :: Very good options for complicated scar conditions
- :: Improved hygienic properties and good disinfectability
- :: Individual bedding options for pressure-sensitive limb regions
- :: Very suitable for skin allergies and sensitive skin
- :: Effective pressure of bony prominences
- :: Silicone socket remains permanently dimensionally stable and unsusceptible to odor development
- :: Very good adhesion properties

Suitable for activity levels 2-4 for everyday prostheses, sports prostheses, and swim prostheses.

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Toe prosthesis in SilPreg® technology

This system is used in the prosthetic care of toe amputation. It is characterized by an excellent skin compatibility and high flexibility, made possible by the use of modern materials.

The design is tailored for each individual patient case. Made from medical grade silicone, it can be simply and easily cleaned and retains its shape. Reinforcements, gel padding and zones of different Shore hardness can be implemented during production. Different color variations are possible to create a detailed replica of the toe.

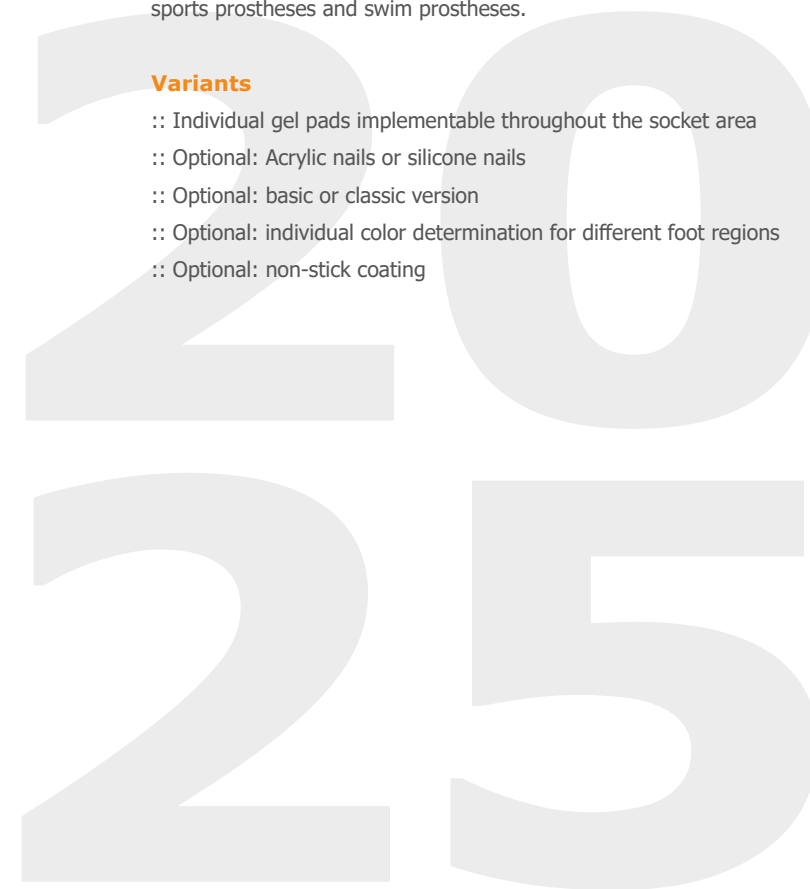
Benefits

- :: Very good options for complicated scars conditions
- :: Improved hygienic properties and good disinfectability
- :: Individual bedding options for pressure-sensitive limb regions
- :: Well suited for skin allergies and sensitive skin
- :: Effective relief of bony prominences
- :: Silicone socket remains permanently dimensionally stable and not susceptible to odors
- :: Very good adhesion properties

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Variants

- :: Individual gel pads implementable throughout the socket area
- :: Optional: Acrylic nails or silicone nails
- :: Optional: basic or classic version
- :: Optional: individual color determination for different foot regions
- :: Optional: non-stick coating

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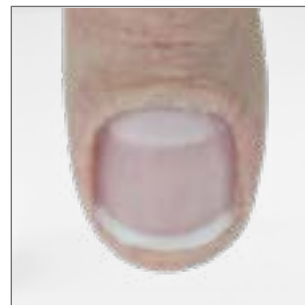
Epithesen

The production of epitheses requires particularly high levels of expertise, because these customized creations are mostly worn visibly. Therefore, it is the goal to adapt the artificial limb as faithfully as possible to the body. The individual skin type, hair, as well as veins must be considered within the multistep process to the finished product. The used material is medical grade silicone, which retains its shape and can be easily and conveniently cleaned. Reinforcements, gel pads and zones of different Shore hardness can be incorporated during manufacturing.

Benefits

- :: Very good fitting options for complicated scar patterns
- :: Improved hygienic properties and good disinfectability
- :: Individual bedding options for pressure sensitive residuals
- :: Very good compatibility with sensitive and allergic skin
- :: Permanently dimensionally stable and not susceptible to odors
- :: Very good adhesion

Variants

- :: Individual gel pads implementable in the entire area of the residual
- :: Optional: Acrylic nail or silicone nail
- :: Optional: Basic or Classic
- :: Optional: individual color determination for different areas
- :: Optional: non-stick coating

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Scar compression

Burns and scals due to thermal, chemical or electrical causes often lead to disordered connective tissue remodeling. As a result, hypertrophic scars develop with hard and unflexible skin. Stiffening of joints due to scar shrinkage can occur especially in cross-articular scars.

Pressure and compression treatment using custom-made silicone keloid masks can help prevent the formation of hypertrophic scars. The mask exert a uniform pressure on the scar and thus ensures a better wound healing process. The silicone keeps the moisture in the skin and makes it more elastic and flexible.

Advantages

- :: Reduction of skin redness in the scar area
- :: Reduces the risk of hypertrophic scars and keloids
- :: Improved hygienic properties and good disinfectability
- :: Minimization of itching in the healing scar
- :: Suitable for patients with skin allergies and sensitive skin
- :: Permanently dimensionally stable and not susceptible to odor

Variants

- :: Individual gel pads can be implemented
- :: Production of full and partial face masks possible
- :: With or without fixation mask

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Contact us!

Many years of experience and continuous development in the field of service production have equipped us with the know-how to flexibly accommodate your special wishes. We discuss directly with you your individual requirements, and we are always guided by our customer's wishes. We will find a customized solution together with you. There are no limits to individual color matching options and creativity. We are your holistic consultant and partner for high-quality service production, and we look forward to closely cooperating with you.

Advantages

- :: Individual color matching
- :: Creative and functional problem solving
- :: Realization of new ideas
- :: Constant customer contact

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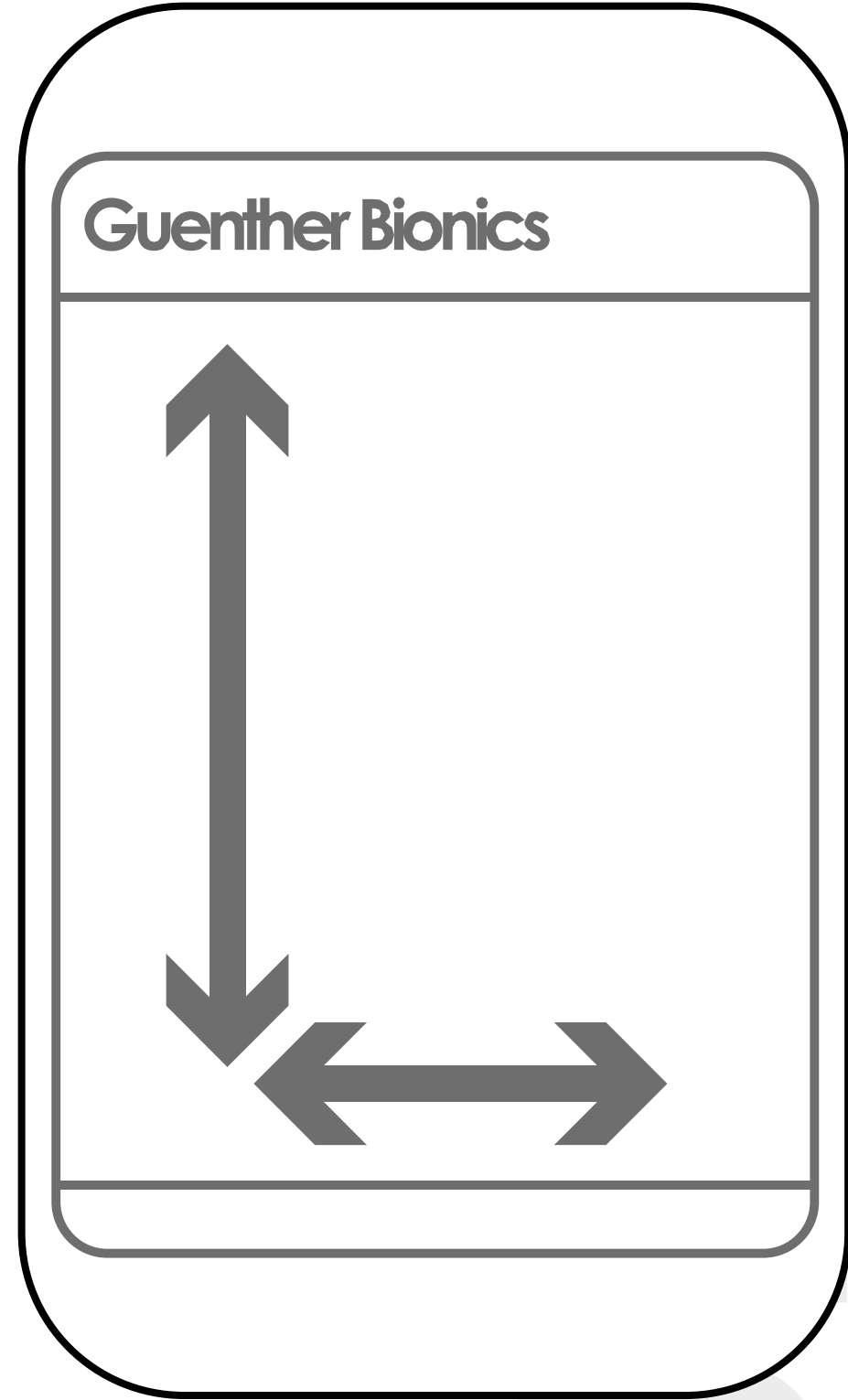
Patient ID: _____

4K03=1 FlexProtector -Standard-
Prothesis side: Left Right
Knee model: _____

4K03=2 FlexCover -Individuell-
 Shaped by Guenther Bionics, not based on foot cosmetics

Prothesis side: Left Right
Knee model: _____

Prothesis height⁽³⁾: _____ **cm**

Prothesis foot height⁽⁵⁾: _____ **cm**

Calf circumference⁽⁴⁾: _____ **cm**

Calf height⁽⁵⁾: _____ **cm**

Pastern circumference⁽⁴⁾: _____ **cm**

Pastern height⁽⁵⁾: _____ **cm**

4K03=3 FlexCover -Premium-
 very precise shaping/reflection of the preserved leg by scanning, can be connected with foot cosmetics

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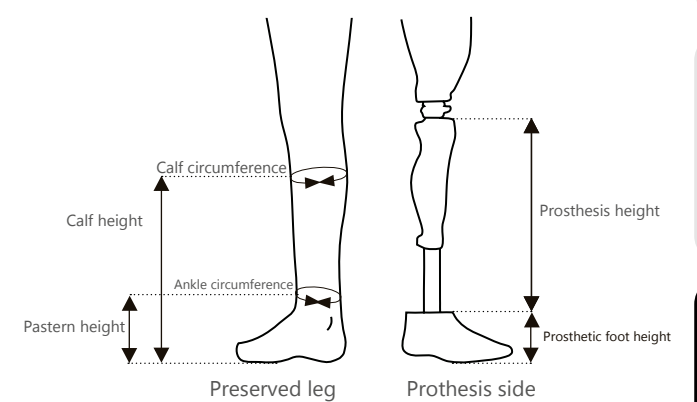
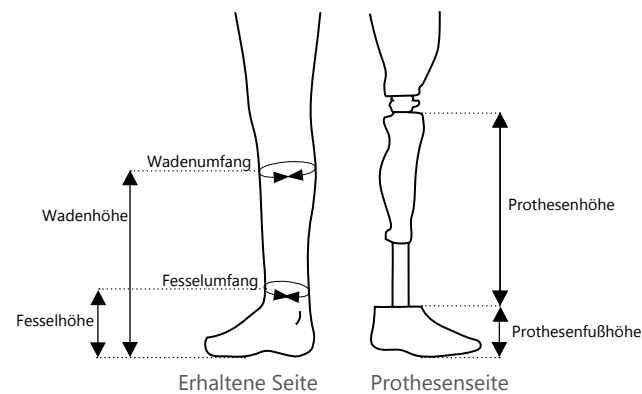
(1) I read the general terms of business and the data protection explanation of the company Guenther Bionics and accept them. (2) Individual color requests are associated with additional costs (3) Measure from the upper edge of the knee prosthesis to the upper edge of the foot. (4) We recommend reducing the size of the cover by about 10% compared to the received leg. (5) Measured on the ground.

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Company: _____ Contact person: _____
 Tel.: _____ Fax: _____

Informationen about the patient

Patienten ID: _____ User-weight: _____ Mobility grade: 1 2 3 4
 Production from digitized data: no yes Date of digitized care: _____

Selection of prosthesis type:

1S010=8 HTV BASIC trans-femoral socket

Order details

Shore hardness selection: firm (65°) medium (50°) soft (35°) standard very soft (20°)

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Non-stick coating **1S01=1**

Digitisation:

Is the socket to be produced to be digitized: yes no

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(1) Existing fittings / dummies (if not ordered) are to be sent in as well (2) I read the general terms of business and the data protection explanation of the company Guenther Bionics and accept them. (3) max. 3 business days from receipt of post at Guenther Bionics / 5 working days for non-stick coating / 25% surcharge (4) Two-tone and production according to RAL specifications are associated with additional costs.

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Company: _____ Contact person: _____
 Tel: _____ Fax: _____

Information about the patient

Patient ID: _____
 Production from digitized data: no yes Date of digitized care: _____

Selection Orthosis type:

- | | |
|---|--|
| <input type="checkbox"/> 1S060=2 Check orthosis forearm hand orthosis made of silicone short | <input type="checkbox"/> 1S061=3 Check orthosis forearm hand orthosis made of silicone long |
| <input type="checkbox"/> 1S060=1 Forearm hand orthosis made of silicone short | <input type="checkbox"/> 1S061=1 Forearm hand orthosis made of silicone long |
| <input type="checkbox"/> 1S060=1A Forearm hand orthosis made of silicone short NON-STICK | <input type="checkbox"/> 1S061=1A Forearm hand orthosis made of silicone long NON-STICK |
| <input type="checkbox"/> 1S062=1 Forearm hand orthosis made of silicone with finger sleeve | <input type="checkbox"/> 1S061=2 Forearm hand orthosis made of silicone without closure |
| <input type="checkbox"/> 1S063=1 Finger correction sleeve made of silicone | |

Order details / Orthosis

Closure:

Color: skin white black blue blue red pink (neon) green (neon) green yellow orange brown

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If the width and number of locks are not specified, the standard version is fitted.

Haptiks silicon:

Matt/structure shiny

Color orthosis:⁽¹⁰⁾

gray black blue orange skin red yellow green purple pink RAL-code: _____

Execution:

<input type="checkbox"/> Standard <input checked="" type="checkbox"/> Orthosis with tab <input checked="" type="checkbox"/> Velcro <input checked="" type="checkbox"/> Reversing loop medial <u>Material orthosis:</u> <input checked="" type="checkbox"/> Shore hardness orthosis 35° ⁽⁴⁾ in 2 mm <input checked="" type="checkbox"/> Reinforcement: (according to markings) 65° ⁽⁵⁾ in 2 mm	<input type="checkbox"/> Custom <input type="checkbox"/> Orthosis with tap <input type="checkbox"/> Orthosis closed <input type="checkbox"/> Velcro <input type="checkbox"/> Orthosis semi-closed (after marked) <input type="checkbox"/> circular, elastic belt <input type="checkbox"/> Reversing loop on the both sides ⁽¹⁾ <input type="checkbox"/> Reversing loop lateral <input type="checkbox"/> Flap (Velcro) ⁽²⁾ <input type="checkbox"/> medial <input type="checkbox"/> lateral <u>Material orthosis:</u> Shore hardness orthosis: <input type="checkbox"/> 20° ⁽³⁾ <input type="checkbox"/> 35° ⁽⁴⁾ <input type="checkbox"/> 50° ⁽⁵⁾ _____ mm Reinforcement: (according to markings) <input type="checkbox"/> 65° ⁽⁵⁾ <input type="checkbox"/> 80° ⁽⁶⁾ _____ mm <input type="checkbox"/> Prepreg Carbon <input type="checkbox"/> Dyneema
--	--

Non-stick coating 1S06=1 Gel pad after marked 1S06=3 Pen holder 1S06=4

Digitisation:

Is the orthosis to be produced to be digitized: yes no

Annotation:

Production time: Standard Express⁽⁸⁾

Place, Date: _____ Signature⁽⁹⁾ / Company stamp: _____

(1) medial + lateral; (2) Velcro in silicone; (3) 20°: very soft; (4) 35°: soft; (5) 50°: firm; (6) 80°: very firm; (8) max. 3 business days from receipt of delivery by Guenther Bionics / 5 business days for non-stick coating / 25% surcharge
 (9) I read the general terms of business and the data protection explanation of the company Guenther Bionics and accept them. **103**
 (10) Two-tone and production according to RAL specifications are associated with additional costs.

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Orthosis pair

Left side

Circumference dimensions	Patient	Plaster Model
Thumb-DIP	cm	Guenther Bionics® GmbH
Heel of the hand (A-dimension)	cm	
Wrist (C-dimension)	cm	
Forearm (C u. D-dimension)	cm	
Forearm (D-dimension)	cm	

Right side

Circumference dimensions	Patient	Plaster Model
Thumb-DIP	cm	Guenther Bionics® GmbH
Heel of the hand (A-dimension)	cm	
Wrist (C-dimension)	cm	
Forearm (C u. D-dimension)	cm	
Forearm (D-dimension)	cm	

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Selection Orthosis type:

- 1S070=3** Check orthosis foot orthosis made of silicone **1S071=1** ICP swimming shoe made of silicone
 1S070=1 foot orthosis made of silicone closed **1S072=1** Toe correction sleeve made of silicone
 1S070=2 foot orthosis made of silicone with closure

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Execution:

<input type="checkbox"/> Standard <input checked="" type="checkbox"/> Orthosis with tab <input checked="" type="checkbox"/> Velcro <input checked="" type="checkbox"/> Reversing loop medial Material orthosis: <input checked="" type="checkbox"/> Shore hardness orthosis 50° ⁽⁵⁾ in 2 mm <input checked="" type="checkbox"/> Reinforcement: (according to markings) 65° ⁽⁵⁾ in 2 mm	<input type="checkbox"/> Custom <input checked="" type="checkbox"/> Orthosis with tap <input checked="" type="checkbox"/> Velcro <input type="checkbox"/> circular, elastic belt <input type="checkbox"/> Reversing loop on the both sides ⁽¹⁾ <input type="checkbox"/> Reversing loop lateral <input type="checkbox"/> Flap (Velcro) ⁽²⁾ <input type="checkbox"/> medial <input type="checkbox"/> lateral Material orthosis: Shore hardness orthosis: <input type="checkbox"/> 20° ⁽³⁾ <input type="checkbox"/> 35° ⁽⁴⁾ <input type="checkbox"/> 50° ⁽⁵⁾ _____ mm Reinforcement: (according to markings) <input type="checkbox"/> 65° ⁽⁵⁾ <input type="checkbox"/> 80° ⁽⁶⁾ _____ mm <input type="checkbox"/> Prepreg Carbon <input type="checkbox"/> Dyneema
---	---

Non-stick coating 1S06=1 Gel pad individual after marked 1S06=3

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(1) medial + lateral; (2) Velcro in silicone; (3) 20°: very soft; (4) 35°: soft; (5) 50°: firm; (6) 80°: very firm; (8) max. 3 business days from receipt of delivery by Guenther Bionics / 5 business days for non-stick coating / 25% surcharge
 (9) I read the general terms of business and the data protection explanation of the company Guenther Bionics and accept them.
 (10) Two-tone and production according to RAL specifications are associated with additional costs.

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Orthosis pair

Right side

Circumference dimensions	Patient	Plaster	Model
	to be completed by OT	Guenther Bionics® GmbH	
Calf (C-dimension)	cm		
Calf approach (B ₁ -dimension)	cm		
Ankle (B-dimension)	cm		
metatarsophalangeal joint (A-dimension)	cm		
Heel (Y-dimension)	cm		

Left side

Circumference dimensions	Patient	Plaster	Model
	to be completed by OT	Guenther Bionics® GmbH	
Calf (C-dimension)	cm		
Calf approach (B ₁ -dimension)	cm		
Ankle (B-dimension)	cm		
metatarsophalangeal joint (A-dimension)	cm		
Heel (Y-dimension)	cm		

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MWK-socket order

SERVICE FABRICATION/SilPreg® TECHNOLOGIE

Order information

Company: _____ Contact person: _____
 Tel.: _____ Fax: _____

Information about the patient

Patient ID: _____ User-weight: _____ Mobility grade: 1 2 3 4

MWK Schaft® serial number: _____ Liner: no yes, _____

Production from digitized data: no yes Date of digitized care: _____

Selection of prosthesis type:

- 1S011=7 Interims-MWK-Socket made of HTV Silikon and plastic
 1S011=6 Diagnostic socket MWK Socket made of HTV silicone and plastic

Liner systems

- 1S011=2 MWK Socket® liner - full contact with liner system
 1S011=3 MWK Socket® swim - full contact, liner system

Vacuum systems

- 1S011=1 MWK Socket® - vacuum full contact
 1S011=3 MWK Socket® swim - full contact, vacuum system

Order details

Shore hardness selection: firm (65°) medium (50°) standard soft (35°) very soft (20°)

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Other built-in systems⁽¹⁾: _____

Adaptersysteme:

- Connection plate 4-hole 3-arm anchor 4-arm anchor
 Pyramid Pyramid receiver

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- Gel pad perineum / individual 1S01=4 Silicone bag perineum / seating area 1S01=6 Boa system
 Reinforcement cap / brace distal 1S01=3 Matrix reinforcement seating area 1S01=2
 Gel pad femur / individual 1S01=5 Non-stick coating 1S01=1

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Selection of prosthesis type:

- | | |
|---|---|
| <input type="checkbox"/> 1S030=3 Check HTV forearm TR | <input type="checkbox"/> 1S050=3 Check socket HTV shoulder disartic. |
| <input type="checkbox"/> 1S030=1 HTV forearm TR | <input type="checkbox"/> 1S050=1 HTV socket shoulder disartic. |
| <input type="checkbox"/> 1S030=2 Definitive TR for forearm (HTV silicone & prepreg carbon) | <input type="checkbox"/> 1S050=2 Definitive socket shoulder disartic. prosthesis (HTV silicone & prepreg carbon) |
| <input type="checkbox"/> 1S040=3 Check socket HTV TH | <input type="checkbox"/> 1S030=5 Armliner Standard with webbing |
| <input type="checkbox"/> 1S040=1 HTV socket TH | |
| <input type="checkbox"/> 1S040=2 Definitive TH for upper arm (HTV silicone & prepreg carbon) | |

Order details

Shore hardness selection: firm (65°) medium (50°) soft (standard) (35°) very soft (20°)

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Locking options between inner and outer socket

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Gel pad individual 1S01=4 Non-stick coating 1S01=1 Boa System

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Company: _____ Contact person: _____
 Tel.: _____ Fax: _____

Information about the patient

Patient ID: _____ User-weight: _____ Mobility grade: 1 2 3 4
 Production from digitized data: no yes Date of digitized care: _____

Selection Prosthesis type:

- | | |
|--|---|
| <input type="checkbox"/> 1S010=6 Check HTV silicone TF | <input type="checkbox"/> 1S010=4 Definitive TF liner
(HTV silicone seat ring & prepreg carbon) |
| <input type="checkbox"/> 1S010=7 Diagnostic socket TF
(plastic + HTV silicone (1S010=1)) | <input type="checkbox"/> 1S010=5 Bathing socket TF-flexible
(HTV silicone with integrated prepreg frame) |
| <input type="checkbox"/> 1S010=1 HTV TF socket | <input type="checkbox"/> 1S020=1 HTV liner for hip disarticulation prosthesis |
| <input type="checkbox"/> 1S010=2 Definitive TF
(HTV silicone & prepreg carbon) | <input type="checkbox"/> 1S020=2 Definitive socket for hip disarticulation prosthesis
(HTV silicone & prepreg carbon) |
| <input type="checkbox"/> 1S010=3 HTV silicone ring TF
(Silicone ring socket entry level) | |

Order details

Shore hardness selection: firm (standard) (65°) medium (50°) standard soft (35°) very soft (20°)

Color silicone⁽¹⁰⁾:

gray black blue orange skin red yellow green lila pink RAL-code: _____

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- | | | |
|--|--|-------------------------------------|
| <input type="checkbox"/> Gel pad perineum / individual 1S01=4 | <input type="checkbox"/> Silicone bag perineum / seating area 1S01=6 | <input type="checkbox"/> Boa system |
| <input type="checkbox"/> Reinforcement cap / brace distal 1S01=3 | <input type="checkbox"/> Matrix reinforcement seating area 1S01=2 | |
| <input type="checkbox"/> Gel pad femur / individual 1S01=5 | <input type="checkbox"/> Non-stick coating 1S01=1 | |

Outer socket:

Dyneema areas (after marked) Open areas (after marked)

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Order details

Shore hardness selection: firm (standard) (65°) medium (50°) soft (35°) very soft (20°)

Color silicone⁽¹⁰⁾:
 gray black blue orange skin red yellow green lila pink RAL-code: _____

Haptiks silicon:
 matt/structure shiny

Color outer socket⁽¹¹⁾:
 standard carbon grey black blue orange skin red yellow green lila pink RAL-Angabe: _____

Made from natural fibers (FOR THE ENVIRONMENT'S SAKE)

Build height:
 maintain shorten: _____ cm Annotation: _____

Locking options between inner and outer socket:

Screw connection (as per indication) Hook & Loop

Components:

Valve system⁽¹⁾: _____ Locking system⁽¹⁾: _____
 Other built-in systems⁽¹⁾: _____

Optional specifications/accessories (please mark on the model)

- Gel pad perineum / individual 1S01=4
- Reinforcement cap / brace distal 1S01=3
- Gel pad femur / individual 1S01=6
- Silicone bag perineum / seating area 1S01=6
- Matrix reinforcement seating area 1S01=2
- Non-stick coating 1S01=1
- Boa system

Outer socket:
 Dyneema areas (after marked) Open areas (after marked)

Digitisation:
 Is the socket to be produced to be digitized: yes no

Annotation: _____

Place, Date: _____ Signature⁽²⁾ / Company stamp: _____

(1) Existing fittings / dummies (if not ordered) are to be sent in as well (2) I read the general terms of business and the data protection explanation of the company Guenther Bionics and accept them. (10) Two-tone and production according to RAL specifications are associated with additional costs. (11) Color coating associated with additional costs.

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Ordering process

FOR THE FABRICATION OF HAND/FINGER PROTESTS

The responsibility for measuring, shaping, color determination and ordering of the prosthesis lies with you as the prosthetist.

- 1. Casting**
Please make an impression of the affected side with the help of alginate or kneading silicone, then pour out with plaster bubble-free.
- 2. Impression of the contralateral side**
Please make a lifelike impression of the contralateral side with the help of alginate or kneading silicone while paying attention to the following points:
 :: The fingers should be in the functional position during the impression procedure
 :: Make sure the wrist does not show any radial and ulnar deviation
 :: The fingers should show a slight flexion
 :: For exact documentation, the dimension sheet and the order sheet for hand and finger prostheses is available. (catalog Guenther Bionics GmbH, page 121/123)
- 3. Shipping to Guenther Bionics**
 :: Shipping of the plaster positive (modeled / not modeled)
 :: Please ensure sufficient padding (Badly damaged plaster models cannot be used for our production process, unfortunately.)
- 4. Preparation and dispatch of the trial prosthesis**
Based on the plaster model, Guenther Bionics will prepare a check prosthesis consisting of a HTV socket and a finger made of sandable kneading silicone (for possible change in shape) and a color ring for later color determination.
- 5. Fitting the check prosthesis**
After receiving the device, you are responsible for checking the fit. Points to watch out for:
 :: Check the socket trim lines
 :: Check fitting (prosthesis length, width, length of fingers, socket volume)
 In case of inaccurate fit, inject HTV-compatible silicone
 :: Check dimension
 Length and width of fingers; maybe toes length; Perform a check fitting to verify proper alignment of the prosthesis. Grind the hand / finger prosthesis into the desired shape or change the shape with the enclosed impression compound; Check the position of the prosthesis, rectify if necessary
 :: Leave the check prosthesis with the user to try out in everyday life (about 7-14 days)
- 6. Color determination and preparation of the photos**

 The illustrations serve as a template for making the photos for color determination. The following points should be noted:
 :: Please make photos under daylight
 :: Please choose white background
 :: Photograph patient standing
 :: For directions see pictures
 :: Closeup of the nails
 :: Color determination with the help of the color pattern ring
 :: Documentation of color on color determination sheet
- 7. Shipping of the modified trial prosthesis**
After the check fitting has been concluded, please send the modified check prosthesis, together with the color sample ring and the modified plaster model back to Guenther Bionics (Am Winterhafen 1, 39114 Magdeburg). Please send the photos by e-mail to the following address, stating the commission and the company: servicefertigung@guentherbionics.de
- 8. Production of the definitive prosthesis**
For all questions regarding functionality, fit and compatibility in everyday life, Guenther Bionics will assume that these were clarified with the help of the check prosthesis. The definitive prosthesis will be produced exactly according to the tried and modified check prosthesis.

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Company: _____ Contact person: _____
 Tel.: _____ Fax: _____

Information about the patient

Patient ID: _____ User-weight: _____ Mobility grade: 1 2 3 4
 Gender: female male User-size: _____ Affected side: left right

Selection of prosthesis type:

- 1S10=1 first check finger Classic or Basic
- 1S101=1 Finger prothesis made of silicone Basic
- 1S101=2 Finger prothesis made of silicone Classic
- 1S102=1 Partial hand prosthesis made of silicone Basic
- 1S102=2 Partial hand prosthesis made of silicone Classic

Selection of nails:

- 1S10=4 Nail standard (one color, from only hinted)
- 1S10=2 Silicone nail for finger prothesis (3-color)
- 1S10=3 Acrylic nail for finger prothesis (3-color)

IMPORTANT:

For fabricaion of the finger or partial hand prosthesis „basic“ and „classic“, the following additions are required:

- Color determination (using the color pattern ring provided by Guenther Bionics) under daylight (see dimension sheet)
- Plaster model
- Photos affected page
contralateral side (Example, page 119, 6.)

Additional information:

Finger joints: freely movable limited range of motion
 Bone end: wide pointed
 Affected fingers: Left hand 1 2 3 4 5
 Right hand 1 2 3 4 5

Annotation: _____

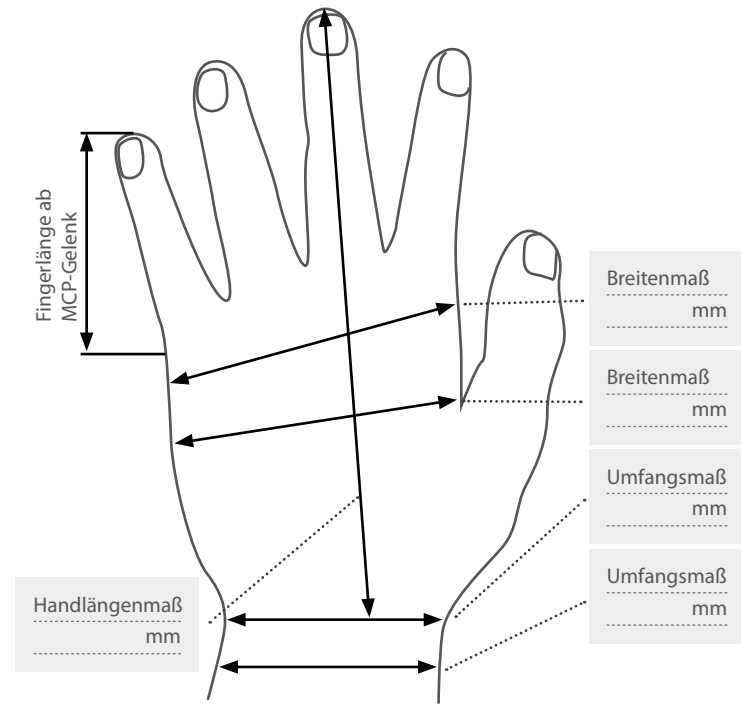
Place, Date: _____ Signature⁽¹⁾ / Company stamp: _____

(1) I read the general terms of business and the data protection explanation of the company Guenther Bionics and accept them.

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
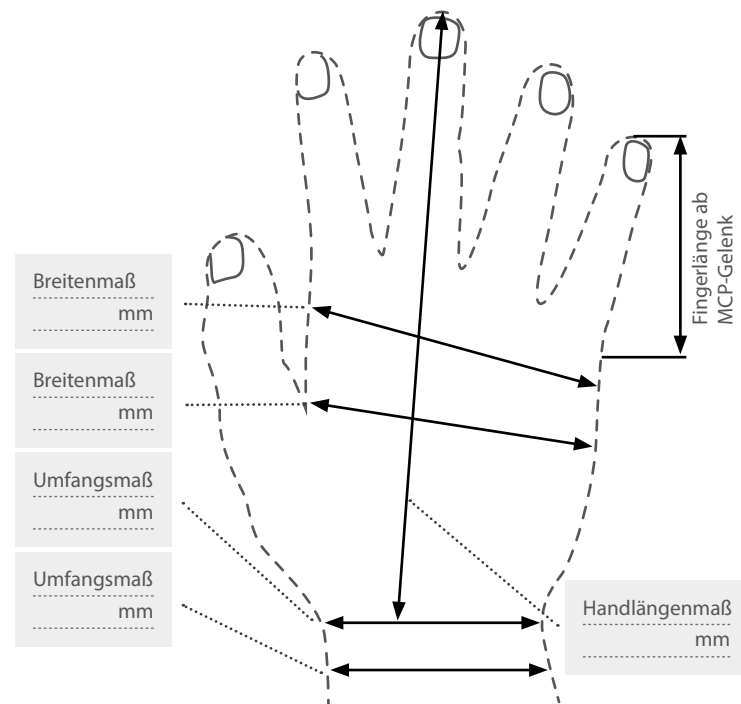
Anmerkung: _____

Silicone part hand / finger prosthesis

DIMENSION SHEET

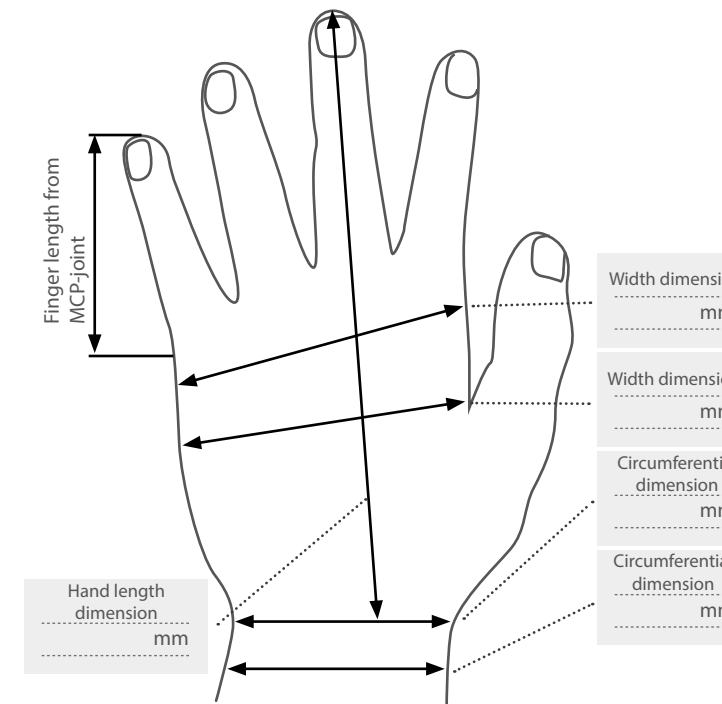
Company: _____ Contact person: _____ Patient ID: _____

Dimension sheet contralateral side

Please draw the circumference dimension of the contralateral side. Take the finger length measurements from the highest point of the MCP joint to the fingertip relaxed functional position.

Finger dimension	I	II	III	IV	V
D
MCP-PIP (Circumference)
PIP-DIP (Circumference)
DIP (Circumference)
Finger length (from MCP-joint)

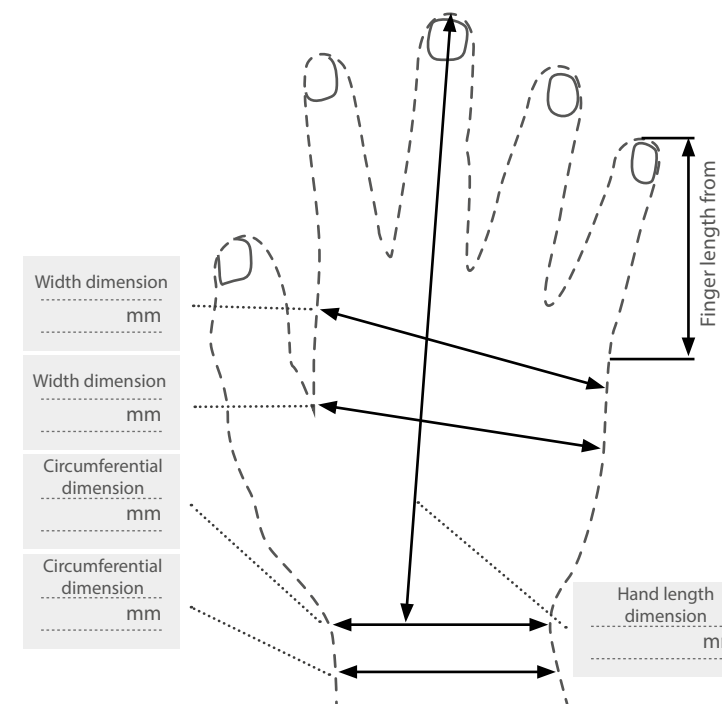Dimension sheet amputated side

Please outline the amputation line.

Finger dimension	I	II	III	IV	V
D
MCP-PIP (Circumference)
PIP-DIP (Circumference)
DIP (Circumference)
Finger length (from MCP-joint)

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
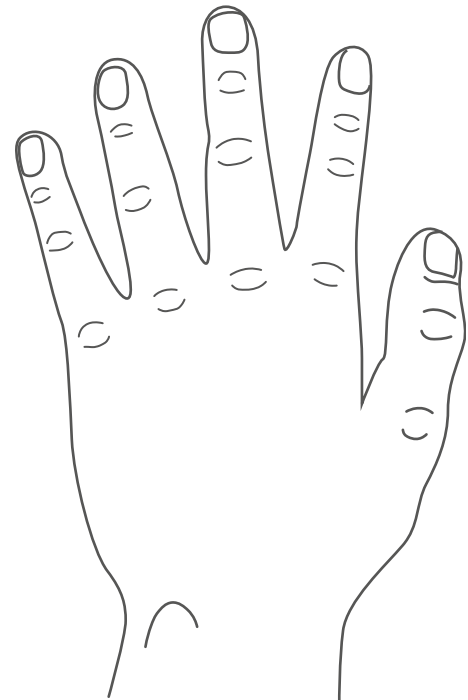
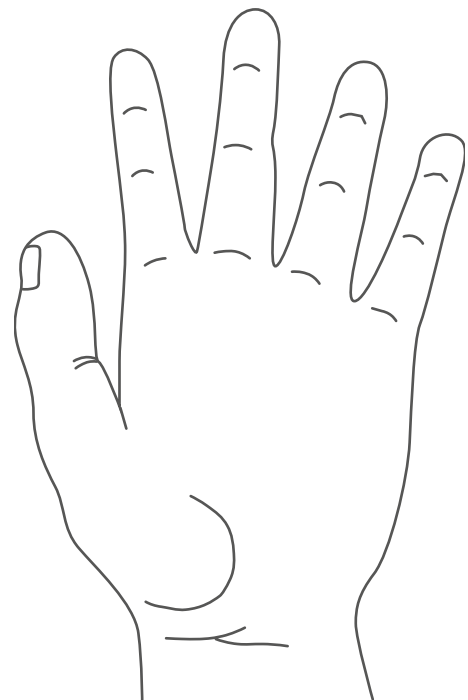
COLOR DETERMINATION SHEET

Company: _____ Contact person: _____ Patient ID: _____

BASIC version > 1 Color | CLASSIC version > up to 3 colors

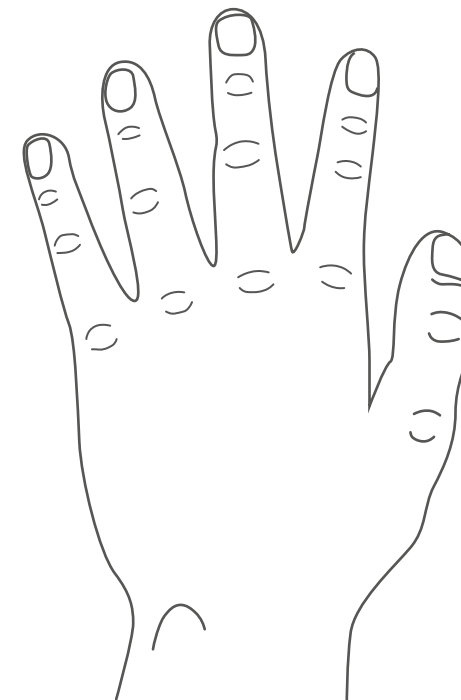Pencil	Color sample
1
2
3
4
5
6
7
8
9
10
11
12
13

Mark the color bar in the sketch and give it a pen number. Enter the corresponding number from the color pattern ring in the table.

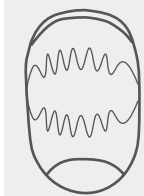
Nails

- Acrylic
- Silicone
- Standard

Nail length

- like photo/Impression on the opposite side
- _____ mm

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Ordering process

FOR THE PREPARATION OF FOREFOOT / TOE PROSTHESES

The responsibility for measuring, shaping, color determination and ordering of prosthesis lies with you as the prosthetist.

1. Casting

- For the plaster cast, please note the following points:
 - :: The impression should be made under load (light pressure on the knee bent at 90°) and extend above the ankle.
 - :: Mark the ankle medially and laterally with indelible marker.
 - :: The dimension sheet and the order sheet for forefoot prostheses is available for accurate documentation. (catalog Guenther Bionics 2018, page 129/131)

2. Impression of the contralateral side

- :: Please make a lifelike impression of the contralateral side with the help of alginate or kneading silicone.
- :: Make sure that the impression is molded in neutral position without contact with the ground.

3. Shipping to Guenther Bionics

- :: Shipment of the plaster positive or, alternatively, the negative
- :: Please ensure sufficient padding (Badly damaged plaster models cannot be used for our production process, unfortunately)

4. Fabrication and shipping of the check prosthesis

Based on the plaster model, Guenther Bionics will prepare a check prosthesis consisting of a HTV socket and a forefoot made of sandable kneading silicone. The check prosthesis is always shipped with a small amount of kneading silicone (for possible changes in shape) and a color ring for later color determination.

5. Fitting the check prosthesis

- After receiving the device, you are responsible for checking the fit. Points to watch out for:
 - Specify the trim lines of the final prosthesis:
 - :: Around the inner and outer malleoli
 - Fit check:
 - :: Perform dynamic and static fitting
 - :: In case of inaccurate fit, inject HTV - compatible silicone (For greater deviations, please change the plaster positive model)
 - Check of dimensions:
 - :: Length and width of the foot
 - :: If applicable toe length
 - :: Grind the forefoot into the desired shape or change the shape with the attached impression material
- Leave the check prosthesis with the user to try out in everyday life (about 7-14 days)

6. Color determination and preparation of the photos

- The illustrations serve as a template for making the photos for color determination. The following points should be noted:
 - :: Please make photos under daylight
 - :: Please choose white background
 - :: Photograph patient standing
 - :: For directions see pictures
 - :: Closeup of the nails
 - :: Color determination with the help of the color pattern ring
 - :: Documentation of color on color determination sheet

7. Shipping of the modified trial prosthesis

After the check fitting has been concluded, please send the modified check prosthesis, together with the color sample ring and the modified plaster model back to Guenther Bionics. (Am Winterhafen 1, 39114 Magdeburg)
Please send the photos by e-mail to the following address, stating the commission and the company: servicefertigung@guentherbionics.de

8. Production of the definitive prosthesis

For all questions regarding functionality, fit and compatibility in everyday life, Guenther Bionics will assume that these were clarified with the help of the check prosthesis. The definitive prosthesis will be produced exactly according to the tried and modified check prosthesis.

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Selection of prosthesis type:

- 1S080=3** First check prosthesis for silicone forefoot prosthesis
- 1S080=4** Second check prosthesis for silicone forefoot prosthesis
- 1S080=1** Silicone forefoot prosthesis basic
- 1S080=1=L** SilAir forefoot prosthesis basic - very light and compressible
- 1S080=2** Silicone forefoot prosthesis classic
- 1S080=2=L** SilAir forefoot prosthesis classic - very light and compressible
- 1S090=3** Check prosthesis for toe prosthesis
- 1S090=1** Silicone toe prosthesis - basic
- 1S090=2** Silicone toe prosthesis - classic
- 1S080=5** Individual gel padding for the forefoot prosthesis (according to drawing)
- 1S090=5** Individual gel padding for toe prosthesis (according to drawing)

Selection of nails:

- 1S08=4 Nail standard (one color, form only hinted)
- 1S08=2 Silicone nail for forefoot prosthesis (3-color)
- 1S08=1 Acrylic nail for forefoot prosthesis (3-color)

IMPORTANT:

For fabrication of the forefoot prosthesis / toe prosthesis „basic“ and „classic“, the following additions are required:

- Color determination (using the color pattern ring provided by Guenther Bionics) under daylight (see dimension sheet)
 - Impression of the contralateral side
 - Photos

Foot:	Frontal view ventral and dorsal medial and lateral Top view of ventral and dorsal
Nägel:	ventrolateral Closeup of the nails
- (Example page 127, 6.)

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

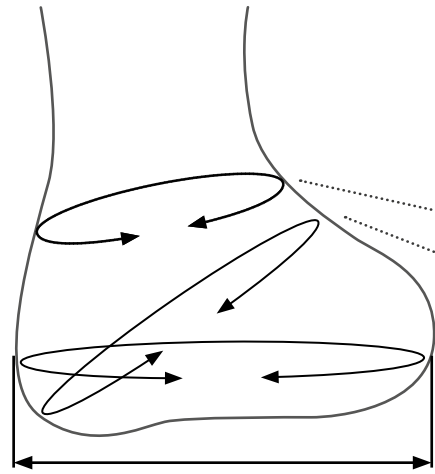
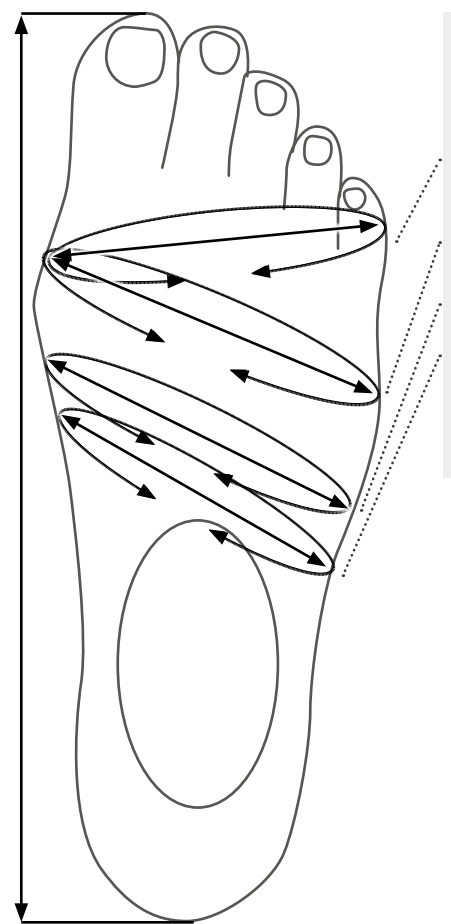
DIMENSION SHEET

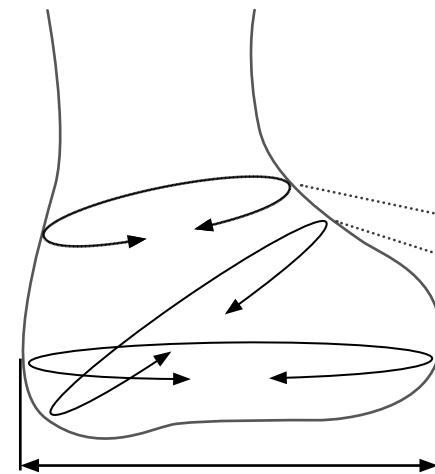Company: _____ Contact person: _____ Patient ID: _____

Measurements (in mm)	Residual limb (in mm)	Plaster	Model
OT	OT	Guenther Bionics®	
m1 width dimension (below lateral ankle)	mm		
Circumference arcuate (below medial and lateral ankle tip)	mm		
Heel-foot-back circumference (h-dimension)	mm		
Horizontal circumference (widest point)	mm		
Total residual limb length	mm		

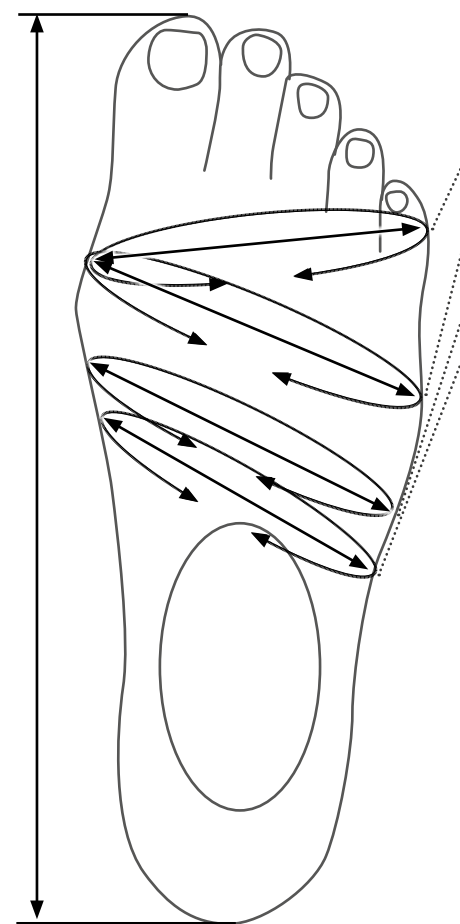	contralateral side		Residual limb (if available)		Plaster	Model
	Circumference	Width	Circumference	Width		
OT	OT	OT	OT	OT	Guenther Bionics® GmbH	
Little toe - Big toe pad						
Little toe pad - Big toe pad						
Pre-instep						
Instep						
Total foot length:						

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


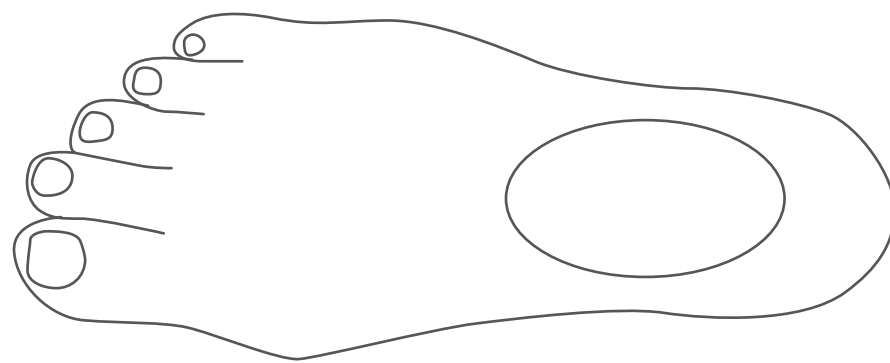
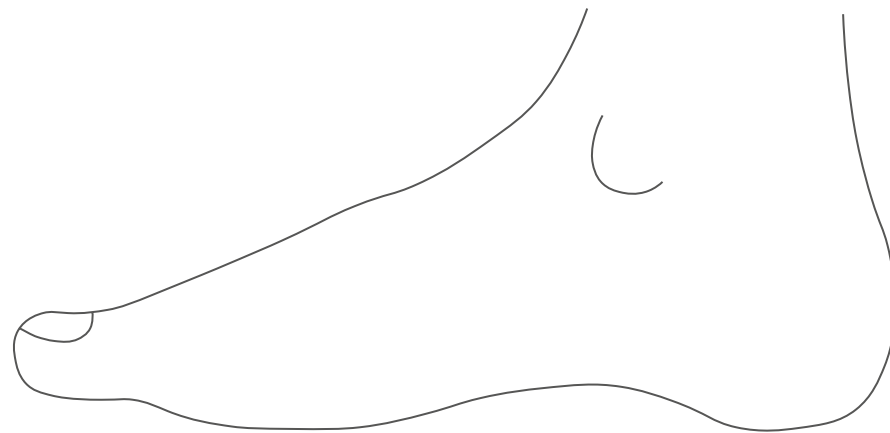
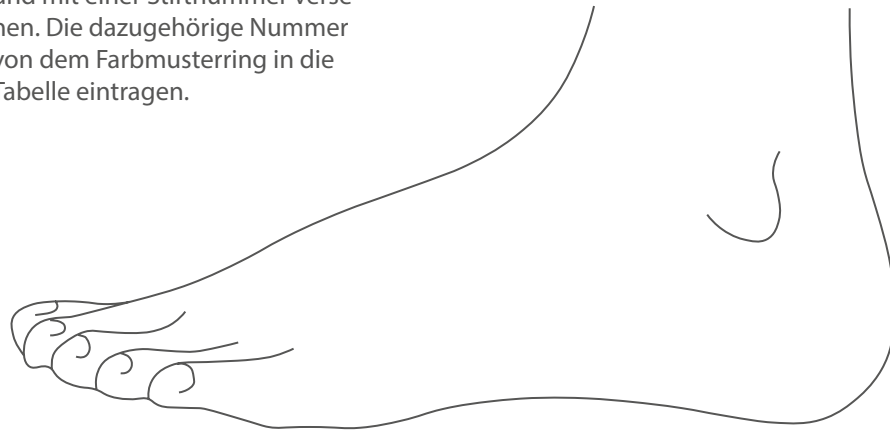
COLOR DETERMINATION SHEET

Company: _____ Contact person: _____ Patient ID: _____

BASIC version > 1 Color | **CLASSIC version > up to 3 colors**

Pencil	Color sample
1
2
3
4
5
6
7
8
9

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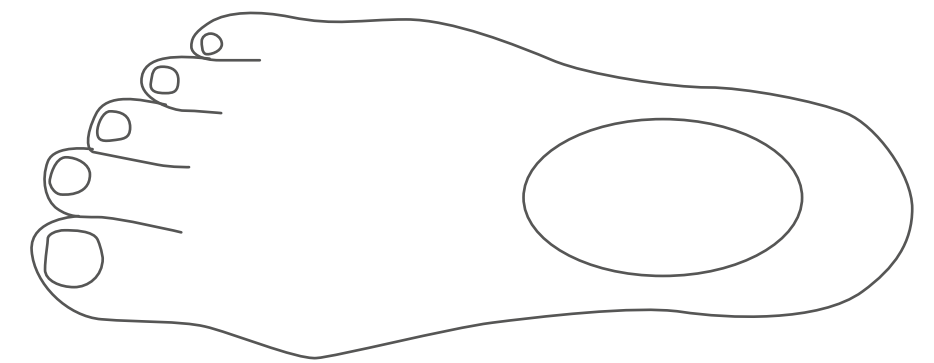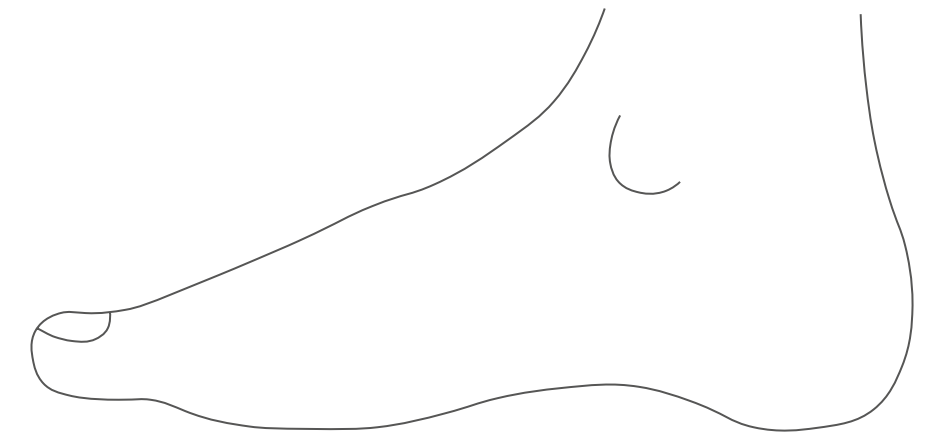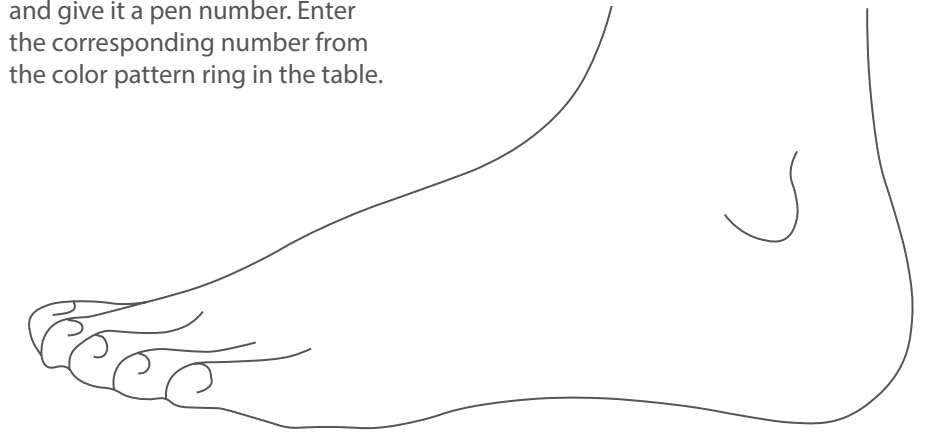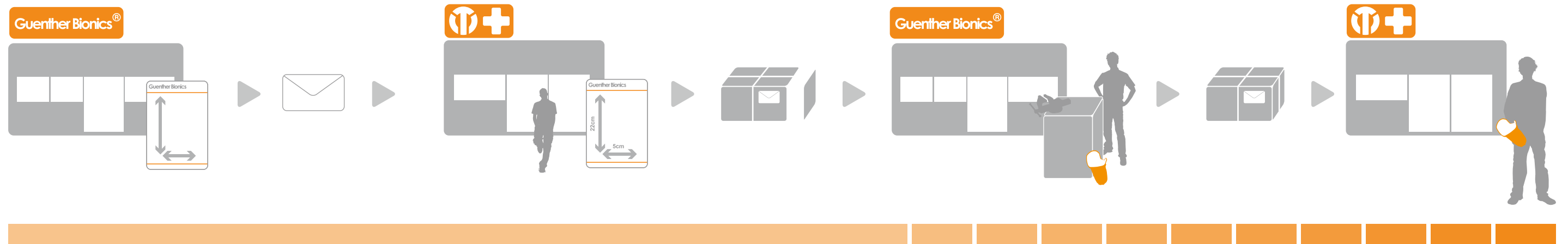
How to order...

Order processing

- :: Plaster positive, -negative or check device
- :: Complete dimension sheet, order form and if necessary color specification sheet

Product manufacturing

- :: According to your requirements
- :: According to the customer's needs
- :: Joint solution finding
- :: Individual color matching
- :: Close cooperation

Production completion

- :: Delivery of the finished product
- :: Quality assessment with form

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